

Irrigate and Dry Quickly and Easily

Root Canal Irrigation,
Suction, Drying System

Canal  Clean

[< See other products](#)




- **Saves significant time and effort required for endodontic treatment**

Canal Clean allows absolute and effective root canal irrigation, suction and drying in the endodontic treatment just by connecting to a dental unit. The effect of water backflow cleans the root canal effectively, suction and drying will be completed in a few seconds.

Irrigation




Water enters the root canal from the outside of the insert tube. When the insert tube starts suction, water backflow occurs in the root canal and removes the residue in the root canal effectively.

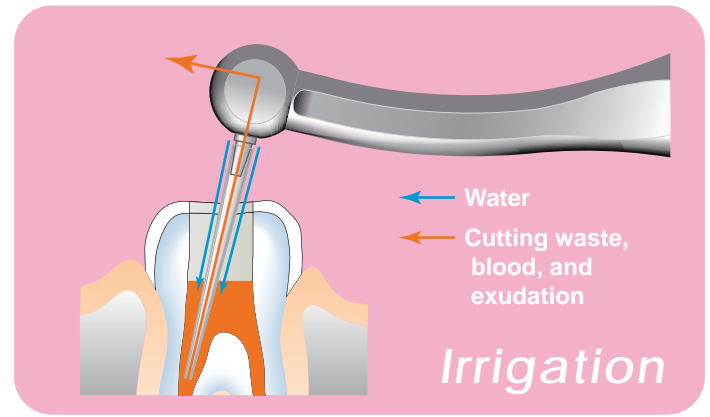

Suction and drying



Stopping water supply allows the insert tube continue suction of residue and remaining water in the root canal. This dries the root canal in just a few seconds.

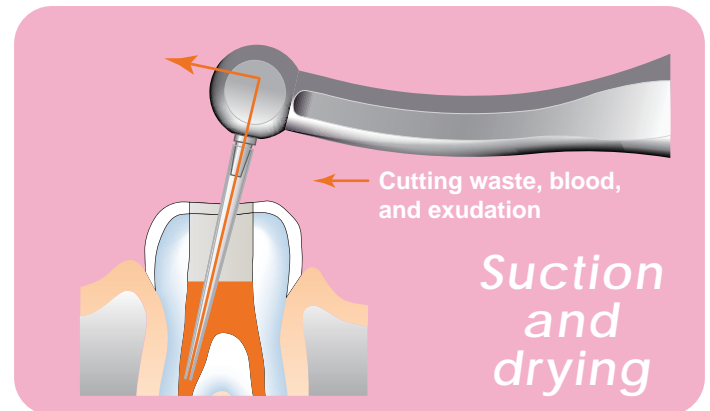

Two choices of tubes - sterilizable and disposable

Insert tube-N
(Flexible)



Canal Clean
-N4H
-N2H

Insert tube-S
(Slim)



Canal Clean
-S4H
-S2H

The insert tubes used in root canals are sterilizable and disposable, it's hygienic. Flexible and slim economical tubes are available.

Easy cleaning and maintenance



When the insert tube is clogged, a push of the cleaning button removes any residue. The quick-disconnect suction head ensures easy maintenance.

Specifications

Model	Canal Clean-N4H / -S4H	Canal Clean-N2H / -S2H
Backend configuration	Midwest 4-hole	Borden 2 or 3-hole
Coupling pressure	0.2 to 0.4 MPa (29 to 58 psi, 2.0 to 4.1 kgf/cm ²)	
Air consumption volume	0 to 30 L/min (Coupling pressure of 0.3MPa)	
Water volume	20 mL/min (Water pressure of 0.2MPa)	
Weight	85 g (3.0 oz)	
Length	152 mm(6.0 inch)	148 mm(5.9 inch)

Specifications are subject to change without prior notice for improvements.

Manufacturer : **MICRON CORPORATION**

2-17-7, Ikegami, Ota-ku, Tokyo, 146-0082, Japan
Tel : +81 3 3755 0396, Fax : +81 3 5747 5396
<http://www.micdent.com/english/> E-mail : office@micdent.com

European Authorized Representative : **EMERGO EUROPE**

Prinsessegracht 20 2514 AP The Hague The Netherlands
Tel : +31 70 345 8570, Fax : +31 70 346 7299